





STRENGTHEN YOUR SINUS SURGERY ARSENAL WITH THE PROPEL FAMILY OF IMPLANTS


The **PROPEL Family** of steroid-releasing implants are designed for use post-ethmoid, frontal or maxillary sinus surgeries, providing physicians a clinically proven solution for their patients.




PACKAGING	IMPLANT	DELIVERY SYSTEM	DELIVERY SYSTEM TIP DIAMETER	IMPLANT NOMINAL HEIGHT	IMPLANT DIAMETER ^{1,2}	SINUS INDICATION	CATALOG #
	PROPEL mometasone furoate implant	Straight	5.2 mm	23 mm	50.0 mm	Ethmoid Sinus	70011
	PROPEL Mini mometasone furoate implant	Curved	4.0 mm	16 mm	42.5 mm	Ethmoid Sinus and Frontal Sinus Ostia	60011
	PROPEL Mini mometasone furoate implant	Straight	4.0 mm	16 mm	42.5 mm	Ethmoid Sinus	60044
	PROPEL Contour mometasone furoate implant	Flexitip	2.8 mm	8 mm	15.5 mm	Frontal and Maxillary Sinus Ostia	50011

 Explore how to load the **PROPEL** Family of implants ...


PROPEL[®]

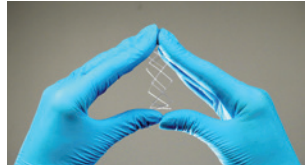


STEP 1
Compress the implant circumferentially into a cylinder using all fingers

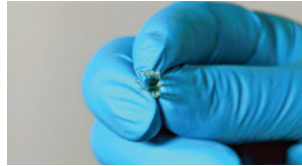

STEP 2
Allow implant to "flower" with 3 fingers to ensure no twisted or crossed struts. Recrimp if cross struts occurs

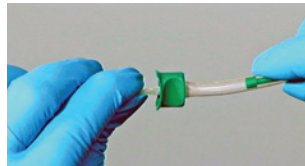


STEP 3
Insert through funnel

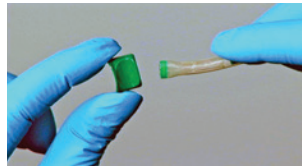


STEP 4
Carefully remove funnel while pinching implant inside delivery system


PROPEL[®]
MINI

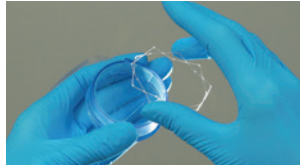


STEP 1
Place implant inside crimping device

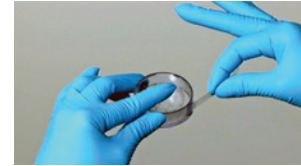

STEP 2
Hover index finger over implant while slowly pulling "zip tie"

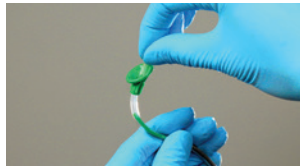


STEP 3
Insert implant into delivery system with funnel facing superiorly (PROPEL Mini SDS funnel mimics that of PROPEL funnel)

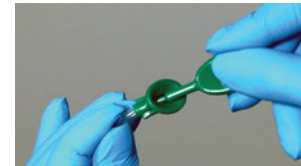


STEP 4
Use loading tool to push through funnel then pinch the applicator tip while removing funnel


PROPEL[®]
CONTOUR

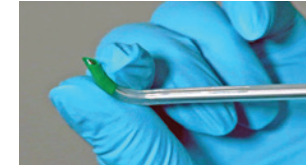


STEP 1
Depress plunger and shape flexible tip to desired angle

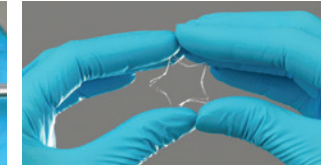

STEP 2
Grasp implant between fingers of both hands and gently compress implant

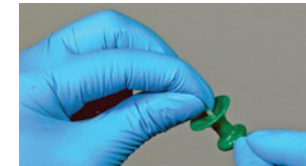

STEP 3
Insert compressed implant into the funnel

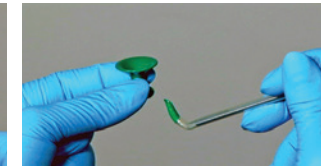

STEP 4
Remove the funnel and the delivery system is ready to be inserted with the tip facing superiorly

- + Load PROPEL sinus implants no more than 3 minutes before use to maximize radial force
- + Do not expose PROPEL implants to fluid or gels prior to use (do not dip in saline)
- + PROPEL implants can be loaded twice, if needed

CUSTOMER SERVICE

Phone: 866-531-6004

Fax: 866-736-1251

Email: CustomerService@intersectENT.com

Web: PROPELOpens.com

For complete instructions, warnings, and precautions, please refer to the instructions for use provided in each implant packaging.

The PROPEL sinus implants are indicated to maintain patency and locally deliver steroid to the sinus mucosa in patients ≥18 years of age after sinus surgery: PROPEL for the ethmoid sinus, PROPEL Mini for the ethmoid sinus/frontal sinus opening, and PROPEL Contour for the frontal/maxillary sinus ostia. Contraindications include patients with intolerance to mometasone furoate (MF) or hypersensitivity to bioabsorbable polymers. Safety and effectiveness of the implant in pregnant or nursing females have not been studied. Risks may include, but are not limited to, pain/pressure, displacement of the implant, possible side effects of intranasal MF, sinusitis, epistaxis, and infection. For full prescribing information see IFU at www.IntersectENT.com/technologies/. Rx only.

References: 1. Data on File, CER R28012, Rev N. 2. Data on File, CR-00023, Rev F.

©2020 Intersect ENT, Inc. Intersect ENT and PROPEL are registered trademarks of Intersect ENT, Inc. in the United States and other countries. MPM-11925 Rev 2.0